
DE SELBSTKLEBEELEKTRODEN
EN SELF-ADHESIVE ELECTRODES
NL ZELFKLEVENDE ELEKTRODEN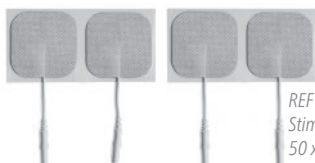

REF 283400
Stimex
50 x 50 mm

REF	Bezeichnung/Type/type	VPE
281000	STIMEX 32 mm rund/round/rond	4
281026	STIMEX 32 mm rund/round/rond sensitiv	4
281027	STIMEX sensitiv 50 x 50 mm	4
281030	STIMEX sensitiv 50 x 90 mm	2
281060	STIMEX 100 x 170 mm	1
282000	STIMEX 50 mm rund/round/rond	4
283000	STIMEX 50 x 130 mm	2
283100	STIMEX 80 x 130 mm	2
283400	STIMEX 50 x 50 mm	4
283600	STIMEX 50 x 90 mm	2
283850	15 x 75 mm	6
4030.0030	SKE Lumbotech. XP	4
4030.0031	SKE Kneehab XP™	4
200365	Druckknopfelektrode/press button	4

DE	EN	NL
-----------	-----------	-----------

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Intended Use

Before using, please read carefully the instructions for use of the self-adhesive electrodes and check the illustrations on the packaging.

Place the self-adhesive electrodes only on clean (for example by washing with soap and water) and grease-free skin. The self-adhesive electrodes have a 2 mm socket resp. a press button. Connect the cable of the device to the socket resp. the press button. **Follow the instructions for use of the electro-stimulation device.**

Lift off the self-adhesive electrode from the carrier foil. To avoid damaging the self-adhesive electrode never tug on the cable. This also applies when detaching the self-adhesive electrode from the skin.

Do not use the self-adhesive electrodes on injured or infected skin surfaces.

The placement of the self-adhesive electrodes is dependent on the indication. **Please comply with the instructions for use of the electro-stimulator in use and follow the instructions of the prescribing therapist.** Always switch the electrostimulator OFF before removing or changing the position of the electrodes.

Caution: Attach the electrodes to the skin so that the electrode surface has even and complete skin contact. **Additionally** make sure that the electrodes are placed at least 2 cm apart from each other.

Therapy must be carried out in accordance with the guidelines of the prescribing therapist. In order to avoid possible skin irritation please remove the electrodes from the skin as soon as the therapy has been completed.

Remove the self-adhesive electrode from the skin after the therapy. Grasp the two plug parts and pull apart in order to remove the connecting cable from the self-adhesive electrode. To avoid damaging the electrode, never tug the cable itself. Any possible gel remaining on the skin is easily removed by washing with soap and water.

After each use, please replace the self-adhesive electrodes to the carrier foil which should then be placed back in the polyethylene bag. Store the bag in a cool location.

The self-adhesive electrodes should not be used at temperatures above 35 °C or below 5 °C.

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